

SNAP N SAVE

The Snap N Save model consists of a seat with attached sliding rings, left and right leg frame, a backrest, two sliding rails, and four legs. This instruction sheet is used for models 37561, 37581, 37591, 37564, 37584, 37594, 37661, 37681, 37691, 37662, 37682, 37692, 37663, 37683, 37693. Weight capacity is 400 lbs for regular length benches only, 350 lbs. for medium length and 300 lbs. for extra long.

Sliding Rail
button

Lock
push
button

Lock
push
button

Sliding
Lock
Red
button

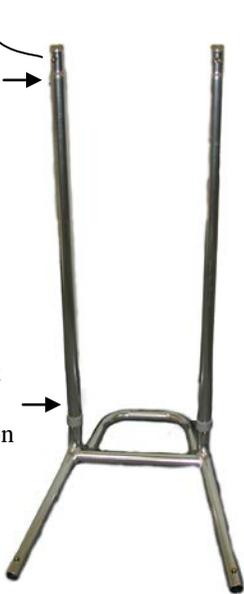


Fig. 1



Fig. 2



Fig. 3



Fig. 4

1. Take the side leg frame with the handlebar on it and put it on the ground (Fig. 1)
2. Determine which side of the transfer bench you want the locks to be on. The side with the handlebar will be the one inside your tub.
3. There are two sliding rails. One of them has 2 brass push buttons and one has 4. Insert the sliding rail with the 4 push buttons on the side you want to have the locks on, making sure that the sliding rail button snaps firmly into the handlebar. (Fig. 1)
4. Put the other sliding rail into the other side, making sure that the button snaps firmly into place.
5. Making sure that the lock side of the seat is on the same rail as the 4 brass push buttons, insert both sliding rails into the sliding rings under the seat on each side. (Fig. 2)

6. Take the other side leg frame and fit it onto the sliding rails as you did with the other side in Step 1. (Fig. 3)
7. Put the bench on the ground horizontally. Press the two brass locking pins on the back brace tubes, then insert into tubes beneath the seat and push in until they click into place. (Fig. 4)
8. Insert each leg into the leg frame and set the same hole location on all four legs. Adjust the suction cups on each leg to fully contact the floor, by turning them.

To Use:

1. Slide the seat all the way to the end outside of the tub (the end opposite to the handle bar). You will hear a click which signifies that the lock has snapped into place. Then transfer the patient onto the seat. For added security, use the safety belt around the patient's lap. To release the locking mechanism, press down on the red button closest to that side and slide the patient with the seat all the way into the tub. When you slide the seat all the way to the other side, you will hear a click which signifies that the lock has snapped into place.
2. After completing the shower, unlock the locking mechanism, by pushing down on the red tab closest to that side and slide the patient with the seat out of the tub all the way to the opposite end. Sliding the patient all the way to the end will lock the seat into place again. Release the safety belt, then transfer the patient off the bench.

For Swivel Seat models only:

To turn the swivel seat, press down on the middle red tab and turn slightly. Once the seat starts swiveling, you can let go of the tab. The seat will lock into place every 90 degrees.

MAKE SURE THE SWIVEL IS LOCKED INTO THE 90 DEGREE POSITION BEFORE SLIDING THE SEAT

To Use the swivel seat model:

1. Slide the bench seat all the way to the end outside of the tub (the end opposite to the handle bar) until the seat clicks into place. Swivel the seat so that the front of the seat is facing the outside of the tub. Transfer the patient onto the seat. For added security, use the safety belt around the patient's lap. Swivel the seat so that the person is facing the shower head. Release the locking mechanism and slide the patient with the seat all the way into the tub until it locks into place.
2. After completing the shower, unlock the locking mechanism and slide the patient with the seat out of the tub all the way to the opposite end until it clicks into place. Swivel the seat so that the patient is facing outside of the tub. Release the safety belt, then transfer the patient off the bench.