


# Revolution Swivel Shower Chair

**PHS3050/PHS3095G**


Assembly and Instruction Manual

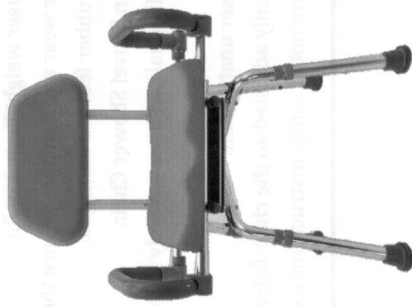

**NO TOOLS required for assembly.**

Remove parts from box and take out of plastic bags.

include: A. One Seat B. One Backrest C. Two Armrests D. Four

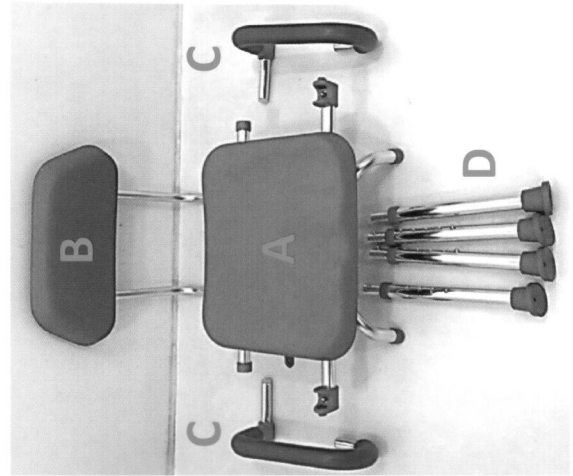



**Step 2.** Align leg to a seat tube and press tension button to insert the leg and slide until the tension button locks in place. Repeat step for other three legs and align evenly. Adjust the rubber feet by rotating them so they are level to the floor.




**Step 3.** Align backrest tubes to rear seat tubes. Press tension buttons on the backrest tubes to insert the backrest into the rear seat tubes and slide until the tension buttons lock in place.



**Step 4.** Align armrest to seat frame tube and push tension button to slide armrest until the tension button locks in place. Repeat the same step on the opposite side.

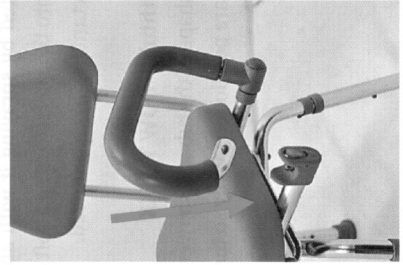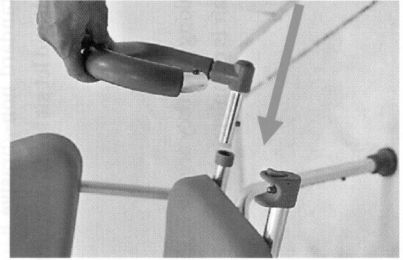

Revolution Swivel Shower Chair is ready for use in a standard shower. Press the red button to rotate the seat. The seat will lock into place every 90 degrees.

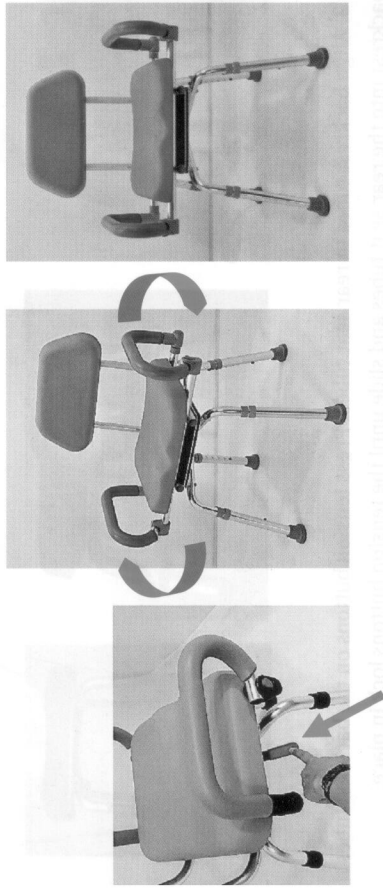

## IMPORTANT SAFETY INFORMATION

### READ ALL SAFETY INFORMATION BEFORE USING

It is the user's responsibility to ensure the client is appropriate prior to using the chair.

- \* Do not exceed the maximum user weight limit 330 lbs.
- \* Inspect at regular intervals for wear and damage. Be sure the plastic parts and rubber feet are in good condition. Do not use if rubber feet are damaged.
- \* Do not stand on the Revolution Swivel Shower Chair.
- \* Never use this device with individuals who are severely mentally, cognitively, physically or vision compromised.
- \* Never leave a compromised user unattended.
- \* Always ensure the legs are evenly adjusted so the chair is level.

NOTE: Failure to follow the above noted safety instructions can lead to injuries.

#### ADJUSTING THE SEAT POSITION:

The height of the Revolution Swivel Shower Chair to the most comfortable position for the user is accomplished by pressing the tension buttons on the legs to raise or lower the seat. All four legs are aligned to the same height to provide a level base. The rubber feet can be adjusted so the chair is level with the floor.

#### DISINFECTION:

The Revolution Swivel Shower Chair can be disinfected by spraying it with any standard non-abrasive hard surface cleaner. Gently wipe the disinfectant onto the surface using a soft cloth and let sit for 10 minutes. Rinse the shower seat to make sure all disinfectant has been removed.

#### LEVELING THE CHAIR:

**Chair Not level in the shower or bathtub? -HOW TO FIX**  
Adjust the feet to accommodate the contour or shape of the tub and shower, making sure they are flat to the floor. Ensure the tension buttons on each leg are positioned evenly so the chair is completely level.