

Parts Reference - Shower Chairs



72231



Part A



Part B

Part C



Part D









Part E



Part F

Assembly Instructions

<p>1. Take leg frame (Part C), and lay it flat on the ground.</p> 	<p>2. Take the two short tubes (Part D), and insert them into leg frame (Part C). Make sure the brass buttons snap firmly into the leg frame.</p> 
<p>3. Take the other leg frame (Part B), and insert the short tubes into the leg frame. Make sure the brass buttons snap firmly into the leg frame.</p> 	<p>4. Set the frame up on its leg tubes. Take the seat (Part A), and place it on top of the frame (with the seat clamps lying on top of the frame). Make sure the curved side* of the seat is on the same side as the protruding backrest tubes (welded under the frame). Firmly snap the four seat clamps onto the short tubes of the frame. The clamps should fit between the gray tube covers. [* If padded seat, the non-rounded-edge side]</p> 
<p>5. Take the 4 legs (Part E), and insert them into the bottom of the leg frames. Set the chair height by using the same button-hole location for each leg. Make sure the brass buttons snap firmly into the leg frame. For your safety, adjust the rubber tips on the legs by turning them so they make maximum floor contact.</p> 	<p>6. Take the backrest (Part F), and press the two brass push buttons on the frame of the backrest. Insert the backrest into the backrest tubes underneath the seat, and push it in until the brass push buttons snap firmly into place.</p> 

Tips for Care and Use

Rubber Tips

Be sure the rubber tips are turned outward for maximum floor contact and stability.

Cleaning

Rinse off the seat area with water after each use to prevent dirt, soap scum, and mineral deposits from forming which could block the push button areas.

If any push button does not completely snap into place, or if you feel the chair is not secure, please do not use the chair, and notify your authorized dealer immediately.

Have Questions? Please contact your authorized dealer for assistance.

Thank you for purchasing an Eagle Health Supplies product!

If you would like to register your product for the 1-year warranty, please fill out the included Warranty Card, and return to Eagle Health Supplies.



EAGLE HEALTH SUPPLIES INC.
www.eaglehealth.com

Instruction Pamphlet

For Shower Chairs

**72131, 72231, 72331
73131, 73231**

Email: sales@eaglehealth.com

Phone: (714) 532 - 1777

Fax: (714) 532 - 9777