

Congratulations on selecting your new Carson magnifier!

With reasonable care, your magnifier will provide you with years of aid and enjoyment. In order to achieve optimum performance, please read the instructions carefully on the proper use and care of your magnifier.

TO USE:

Place the magnifier over the object you are viewing. The higher the magnifying power, the closer the magnifier must be to the object to get a proper focus - see chart below for examples. For best results, your eye should also be reasonably close to the magnifier.

MAGNIFYING POWER	APPROXIMATE FOCAL LENGTH (in.)
2x	10
3x	5
6x	2
10x	1

This guideline also applies to spot lenses, which typically require the magnifier to be quite close to the object. If your magnifier has a Fresnel lens (flat lens with grooves), the above guidelines still apply, so that even though the lens is flat, the magnifier will not focus if placed directly on top of an object.

** Clean using microfiber lens cloth. Do not use any abrasives, chemical cleaners, or solvents.*

CARSON®

Still have questions? Call us toll free at 1-800-9-OPTICS or 631-963-5000