

# SHOE HORN & SOCK REMOVER



## INSERT

Insert shoe horn and slide foot down



## SLIDE

Slip shoe horn between sock and leg



## PUSH

Push sock down over heel and off your foot

hook facing outward

# DRESSING HOOK



## REACH

Extend your reach to pick up clothing and other items



## PULL

Use the hook to assist in pulling clothing on and off

easy™  
TO USE PRODUCTS


reach  right

# COMPRESSION SOCK TOOL USER GUIDE


## ASSEMBLY INSTRUCTIONS:

### EXTENSION & HANDLES

Insert end into socket and lower

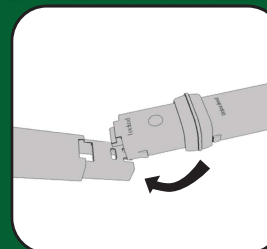


Slide locking ring forward until click

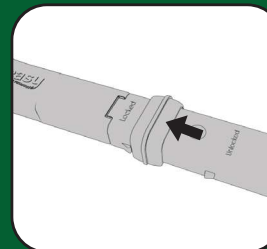


### PADDLES & CANISTER

Align slots onto clips and pull until handles lock

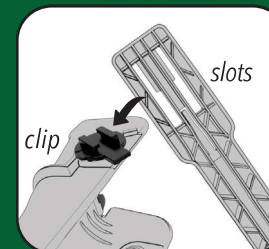

To detach: press lock release and push handle off

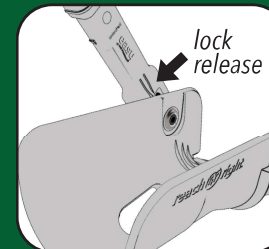

VIEW INSTRUCTIONS AND DEMOS ONLINE  
[WWW.REACHRIGHTTOOL.COM](http://WWW.REACHRIGHTTOOL.COM)

# SOCK CANISTER SOCK AID

**STOP STRAINING...  
REACH RIGHT!**



## ATTACH

Connect paddles to black clips with tabs pointing toward handles



## POSITION

Hold sock canister between legs with handles extending out




## LOAD

Stretch top of sock over canister and slide down until you see inside of toe



## INSERT

Slide foot into sock until foot reaches the toe of the sock

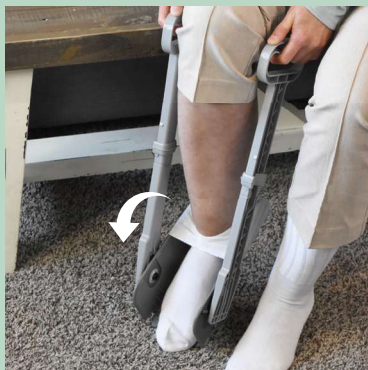


## ROTATE

Rotate sock canister up until grips engage with floor then push heel into sock



## PULL

Use handles to pull sock up leg and push foot down to the floor

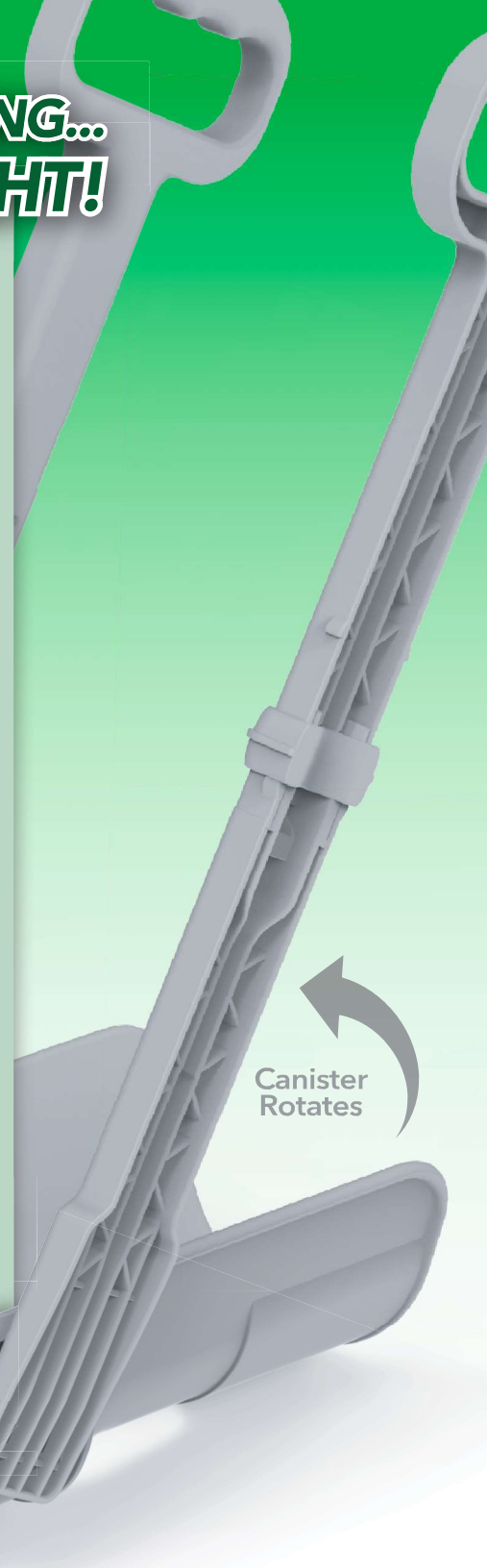

SEE HOW TO ASSEMBLE & USE AT  
[WWW.REACHRIGHTTOOL.COM](http://WWW.REACHRIGHTTOOL.COM)